


Measurements Are Required

To take circumference measurements, (C1 thru C4)

Use a string or something soft and flexible

Measure around the penis (like a belt goes around your waist).

Lie string flat on a ruler and note the size.

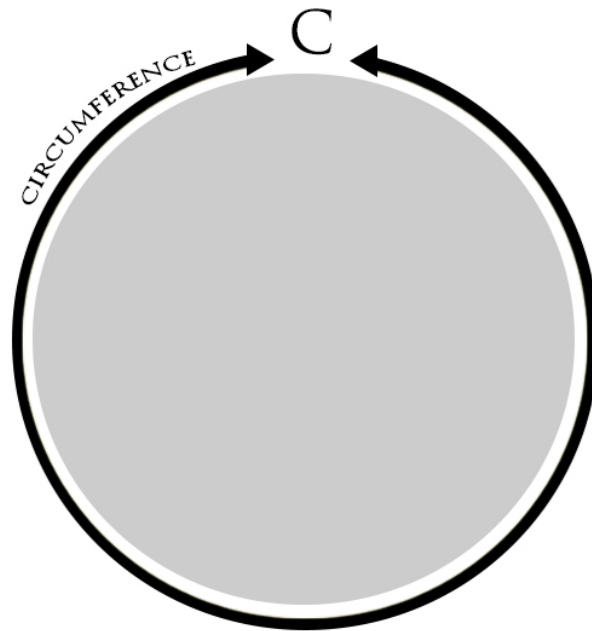

Please furnish all measurements in inches, accurate within 1/4 inch.

Step 1: I can get an erection easily. If no, proceed to Step 2.

- Provide full set of measurements for Erection.
 - C1= Around the head of penis at biggest point.
 - C2= Where the head meets the shaft.
 - C3= Half way down the shaft.
 - C4= Shaft one inch from the body.
 - L = Base to tip along the top.
- Skip Step 2 and 3 and proceed to order.

Step 2: I can get erections, but not every time. If no, proceed to Step 3.

- Provide full set of measurements for Erection.
 - C1= Around the head of penis at biggest point.
 - C2= Where the head meets the shaft.
 - C3= Half way down the shaft.
 - C4= Shaft one inch from the body.
 - L = Base to tip along the top
- Provide C4 Measurement only for Flaccid
- Please note: If there is more than 3/4" difference between C4 erect and C4 flaccid you might consider adding a second receiver.
- Skip Step 2 and 3 and proceed to order.

Step 3: Erections are not possible.

- Provide the C4 Measurement flaccid (Shaft one inch from the body).
- Give us your best possible length measurement
- Proceed to order.